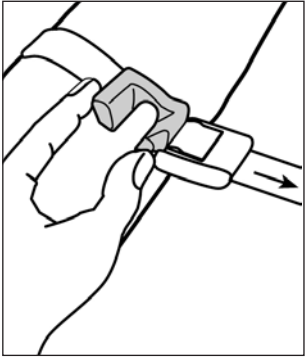
Easy-Verwendung mit Venenfixierung
"Easy application" with vein fixing
Utilisation "Easy" avec fixation de la veine
Bruksanvisning för Easy-användning, venen fixeras

Stauer anlegen: Am Punktionsort anlegen. Easy-Teil im Schloß hörbar einrasten
Applying the tourniquet: Apply at puncture site. Snap the "Easy" part audibly into position in the lock.

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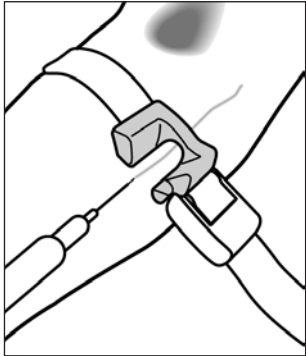
Fixing the vein: Apply the open channel parallel to the vein to be punctured and fix with one hand. Using the other hand, tighten the tourniquet at the end of the strap. If necessary, adjust the "Easy" part.

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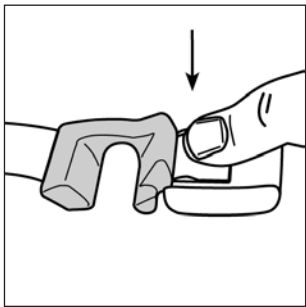
Puncturing/ inserting cannulae: Puncture just in front of the fixing channel. It is possible to inject/take blood with the tourniquet applied due to the open congestion channel (even without reducing the tourniquet pressure).

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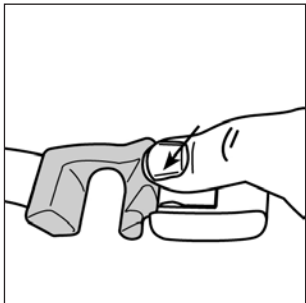
Reducing the pressure: Gently remove the pressure by pressing the button on the metal lock.

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Releasing: Release the strap by pressing the "Easy" part.

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Conventional application (without vein fixing): Apply the tourniquet to upper arm and snap the "Easy" part audibly into position in the lock. Tighten strap. Puncture. Gently reduce the pressure before injecting/ taking blood.

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Benefit from the advantages of the Easy tourniquet 904

- Problem veins are easier to puncture
- Infusions are simple to apply
- Both hands are left free for puncturing
- Multiple punctures at the same extremity are possible
- Measured pressure reduction and release using only one hand
- Based on tried and trusted PRÄMETA technology

The new special element fixes the vein to be punctured. In this way, problem veins are no longer able to evade and infusions are applied more securely.

Congestion at the puncture level is a new feature.

The selected vein is not congested but is easy to puncture due to the increased flow and volume. This means that the injection enters the circulation immediately. It is also possible to take blood without reducing the pressure - no risk of haemolytic serum.

The Easy tourniquet by PRÄMETA:

The new additional solution for problem cases.

For normal applications, can also be used like a conventional tourniquet.

The strap is autoclaveable at 1 bar up to 120° C/250° F.

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